Comfort Cushion Liner Comfort Locking Liner

Instructions for Use

CTTCP22L-CTTCP40L
CTTLP22L-CTTLP40L

EN	Instructions for Use	1
FR	Instructions D'Utilisation	5
DE	Gebrauchsanweisung	9
IT	Istruzioni per L'Uso	14
ES	Instrucciones de Uso	19
NO	Bruksanvisning	24

1 Description and purpose

EN

Application:

These instructions are for use by the Clinician/Practitioner.

The Blatchford Comfort silicone liner provides a cushioned socket interface and is manufactured from biocompatible materials. It is intended for a low to moderate activity user. Please read these instructions carefully before fitting the liner.

To optimise comfort ensure that the user is instructed in the correct way to handle and don/doff the liner. Also ensure that they are made aware of how to maintain the liner and to keep it clean for maintaining hygiene as set out in these instructions.

The user should be advised to contact their clinician if their condition changes.


2 Product selection and fitting

To determine the correct size

It is important that the correct size is specified to ensure proper fit and comfort in use.

Measure the circumference 4cm up from the Distal end. If an exact size is measured choose the next size down.

Sizing Example:

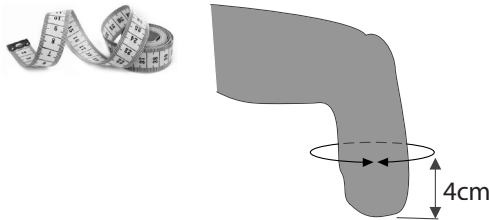After donning the liner allow the patient to wear the liner for approximately 10 minutes. If any numbness or tingling is felt remove the liner and when normal sensation returns try the liner again if numbness occurs again do not use the liner.

If the measured size is an exact match for a Liner size choose the next smaller size to ensure a snug fit for the Liner.

Wearers with sensitive skin, diabetics and vascular cases should be extra vigilant and may need to apply lubricant to sensitive areas. We recommend a routine visual check and if required the user should consult their medical practitioner.

Blatchford Comfort	Size (cm)										
	22	23.5	25	26.5	28	30	32	34	36	38	40
Transtibial											
Cushion	✓	✓	✓	✓	✓	✓	✓	✓	✓		✓
Locking	✓	✓	✓	✓	✓	✓	✓	✓	✓		✓

Order Example:

To Specify Liner Model:

Comfort [C]

Trans-Tibial [TT]

Cushion [C], Locking [L]

Parallel [P]

Size 22-40

Light Tone [L]

Available from size 22 to size 40:

CTTCP22L to CTTCP40L

CTTLP22L to CTTLP40L

C	TT	C	P	26	L
Blatchford COMFORT	Trans Tibial	Cushion	Parallel	Size 26.5	Light Tone

e.g. CTTCP26L

Trimming

The liner may be trimmed to suit as required.

We recommend the use of a rotating disc type cutter to achieve a 'rolled' or bevelled edge.

Trim the liner to allow a suitable overlap beyond the hard edge of the socket. The liner may be trimmed in a 'waveform' profile to minimise the risk of edge effect.

 Where possible try to avoid abrupt changes of contour and sharp edges which could cause tears in the silicone and fabric.

Take care when handling the Liner to avoid possible contamination with materials such as fibreglass which will stick to the Liner and cause skin irritation.

Locking

When using a locking liner the integrated matrix should finish just below fibula head/patella tendon level. The pin should be secured using Loctite 411.

If fitting with a shuttle lock ensure that the user has a socket with distal end contact.

The shuttle lock must always follow the long axis of the residual limb.

Donning and doffing


Care must be taken during donning/doffing not to damage the liner with fingernails or sharp jewelry.

Do **NOT** pull or stretch the liner.

1. **Roll** the liner inside out.
2. Make sure the liner is positioned correctly to suit the top profile.
3. **Roll** the liner on taking care to expel any trapped air.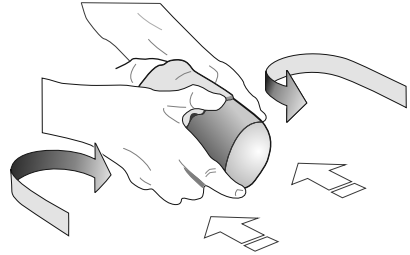

Warning: when donning a sock, clothing and the prosthetic limb, be aware that the liner can build up a static charge.

3 Cleaning and care

Cleaning and care

Wash the inside of the liner daily to avoid any build up of bacteria. Use a simple, unperfumed soap (ideally pH balanced).

Hand wash or Cool machine wash.

Rinse well in clean water to remove all residues and dry thoroughly before use.

Take care when drying and handling the liner. Pat dry with a lint-free cloth or leave to air dry. Do **NOT** leave the liner inside out to dry as this can stretch the liner and permanently distort the shape.

Take care handling the liner when it is inside out to avoid picking up dust, grit and other contamination which could cause irritation to the skin.

Skin care

It is recommended that when using the liner the residuum should be washed daily with a simple, unperfumed soap (ideally pH balanced). Rinse thoroughly to remove any soapy residue before drying thoroughly. Use a lotion for dry skin. Make sure that any damaged skin is properly and suitably dressed.

DO NOT tumble dry.

Do **NOT** use alcohol sprays, household cleaners or abrasives which could damage the liner and cause skin irritation.

Precautions

Warning: To avoid danger of suffocation, keep the liner away from babies and children

Keep away from direct heat sources which could damage the liner.

Stop using the liner if it is torn or damaged.

For other medical conditions please check with your physician or medical practitioner for following this recommendation for skin care.

Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by the component combinations that were not authorized by the manufacturer.

CE Conformity

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class 1 Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

Warranty:

The liner is warranted for 6 months from original date of purchase unless otherwise stated. See Blatchford catalogue for details.

Environmental Aspects

This product is fabricated from silicone rubber and fabric that cannot be easily recycled: please dispose of it responsibly as general waste, as per local waste handling regulations.

Manufacturer's Registered Address

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Blatchford SAS
Endolite India Ltd